

**REF** V7149, 32393B5, 32393B3, 32393B2, 32393B4, 32393B1

**SurgiVet**<sup>TM</sup>

## For Veterinary Use Only



Caution



Catalog Number



Manufacturer



Not made with natural rubber latex



CAUTION: Federal (USA) law restricts this device to sale by or on the order of a veterinarian

PRODUCT CODE	DESCRIPTION	MASK DIMENSIONS (WITHOUT CONNECTOR)	REPLACEMENT DIAPHRAGM
32393B5	Small Feline Mask	50.8 mm W x 25.4 mm H (2 in. W x 1 in. H)	32368B5
32393B3	Medium Feline Mask	63.5 mm W x 31.75 mm H (2 1/2 in. W x 1 1/4 in. H)	32368B4
32393B2	Large Feline Mask	95.25 mm W x 54 mm H (3 3/4 in. W x 2 1/8 in. H)	32368B2
32393B4	Small Canine Mask	95.25 mm W x 76.2 mm H (3 3/4 in. W x 3 in. H)	32368B2
32393B1	Large Canine Mask	139.7 mm W x 114.3 mm H (5 1/2 in. W x 4 1/2 in. H)	32368B1
V7149	Anesthesia Mask Kit	Includes one each - 32393B5, 32393B3, 32393B2, 32393B4, 32393B1	*

\* See individual mask for replacement diaphragm.

## Anesthesia Masks

### Description

The SurgiVet<sup>®</sup> Anesthesia Masks are used to deliver supplemental oxygen and anesthetic agents to patients that require sedation or oxygen therapy. These are sold individually and in a kit.

The mask has a 15 mm adapter allowing the attachment of a standard anesthesia machine breathing circuit. The diaphragm is easily removed for cleaning or replacement.

### Instructions for Use

1. Connect an anesthesia machine breathing circuit to the 15 mm adapter on the mask.
2. Apply the mask to the patient's nose and mouth, creating a seal.
3. Set the appropriate O<sub>2</sub> and/or appropriate anesthetic flow rate for the patient.

**NOTE!** For long term insufflation, oxygen should flow through a bubble type humidifier to prevent mucosal drying.

### Cleaning Instructions

The masks should be cleaned after each use with dilute chlorhexidine solution or mild soap and water. For the best method of cleaning, remove the diaphragm and 15 mm adapter from the mask. Then dry the parts and reinstall the diaphragm and adapter for the next use.

**WARNING!** Do not autoclave.

For additional information, contact Veterinary Clinical Support at 1 214.618.0218 or toll free in the USA at 1 800.258.5361.

Manufactured for:

**Smiths Medical ASD, Inc.**  
6000 Nathan Lane North  
Minneapolis, MN 55442 USA  
Tel: 1 800 258 5361 (US/CA)  
Tel: +1 614 210 7300

SurgiVet and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009, 2011, 2019 Smiths Medical. All rights reserved.

www.smiths-medical.com

**smiths medical**